


Face Shield Assembly Instructions

Part Number 400612 | Tested to ANSI Z87.1-2015 Standard, Splash only

EcoStar face shields are personal protective equipment used to protect the facial area from splashes, sprays and spatter of body fluids. Replacement parts can be ordered by emailing sales@ecostarllc.com.

Package Includes:

- A. Clear plastic PETG shield (pn 400612_1)
- B. Blue plastic headpiece (pn 400612_2)
- C. Blue plastic bottom piece (pn 400612_3)
- D. Adhesive foam comfort strip (pn 400612_4)
- E. Crescent foam cover pad (pn 400612_5)
- F. Elastic band (pn 400612_6)

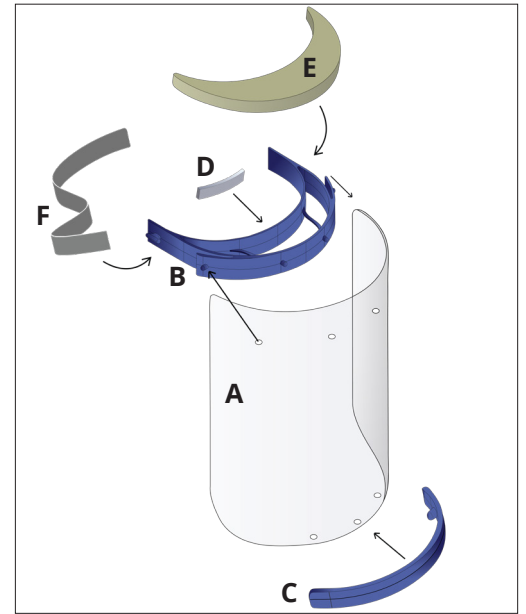

Assembly Instructions (NOTE: Wash your hands and surroundings before assembly. Wipe down with an alcohol wipe before use.)

1. Mount the clear plastic shield on the blue plastic headpiece by placing the first hole on the top of the shield on the first pin of the headpiece.
2. Flex the clear plastic shield to connect the second hole of the shield to the second pin of the headpiece.
3. Flex the clear plastic shield to connect the third hole of the shield to the third pin of the headpiece.
4. Push the back of the fourth pin and straighten headpiece to meet the last hole on face shield.
5. Attach blue plastic bottom piece to the outside of the bottom of plastic shield by repeating steps 1-3.
6. Add adhesive foam comfort strip to blue plastic headpiece by peeling off paper to reveal adhesive strip (optional).
7. Insert crescent foam cover pad into blue headpiece.
8. Attach elastic band to headpiece by placing the buttonhole in elastic band on tab of headpiece.

